Pupillendilatator *Typ 5S*

Pupil Dilator *Type 5S*

MORCHER[®] IOL

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The Morcher Pupil Dilator Type 5S

The optimal temporary implant for use with patients who have cataract and contracted pupils.

Properties:

The pupil dilator type 5S is a semicircular elastic PMMA ring for the expansion of the pupil during phacoemulsification. It is supplied in sterile packing for a single use.

Indication:

Expansion of the pupil during phacoemulsification for patients who have pupils which will not dilate adequately.

Important Note:

The pupil dilator should be implanted only with visco-elastic protection of the corneal endothelium.

Advantages:

The pupil dilator type 5S can be used with any type of incision including corneal and scleral tunnel approaches.

No additional incisions are needed. Its insertion and removal can be performed with ordinary surgical instruments. It provides physiological stretching of the pupil.

Contraindication:

The pupil dilator 5S should be used only with phacoemulsification. Do not use this temporary implant for delivery of the nucleus in extracapsular surgery.

Our recommendation:

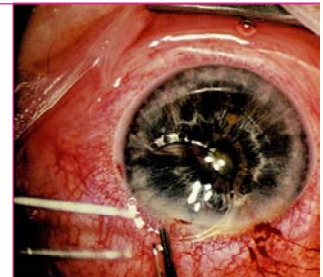
For implantations of the pupil dilator using an injector, we recommend the Geuder model (type G-32970).

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Surgical Procedure without injector

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
Deepen the anterior chamber well with visco-elastic. The ring of the pupil dilator type 5S should be held vertically so that the tip can enter through the wound into the anterior chamber easily. As the tip is advanced into the pupil the dilator must be rotated so that the ring lies in the plane of the iris. The tip should engage the edge of the pupil at 6:00. As a rule, this happens almost of its own accord.

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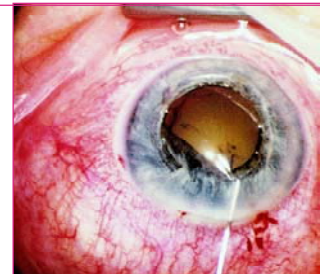Maintaining the depth of the anterior chamber with a visco-elastic, the semicircular arms of the dilator are then attached one by one onto the edge of the pupil using implant hooks. The stretching of the pupil may result in temporary distortions.

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
The gap in the ring should rest next to the phaco incision.

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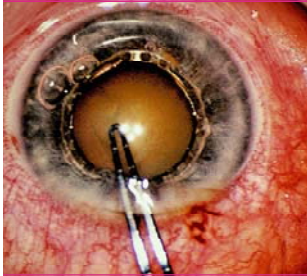
4

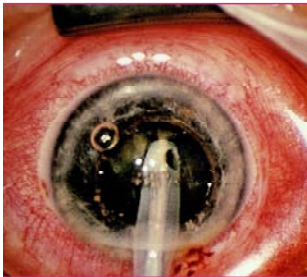
After positioning the pupil dilator, the pupil will usually be 5.0 to 6.0 mm in a diameter.

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5

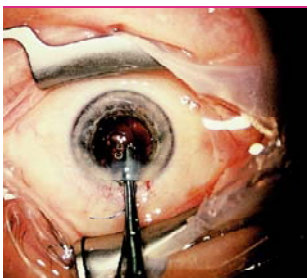Phacoemulsification with the ring in place can now be performed conveniently.

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6

The pupil dilator may remain in place for the implantation of a foldable or small PMMA IOL. Remove the pupil dilator if an inflexible PMMA posterior chamber lens with an optic larger than 5.5 mm is to be implanted.

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7

To remove the pupil dilator, unhook its arms one at a time with an implantation hook, with a depth of the anterior chamber maintained with a visco-elastic. Draw one end of the pupil dilator into the wound, and slide out the entire device.

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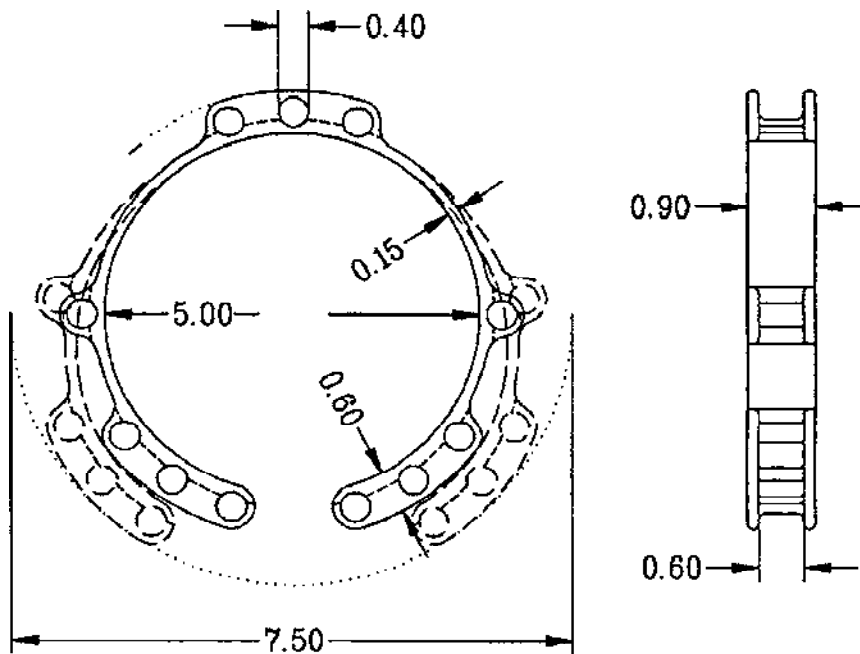
Important!

After removing the pupil dilator type 5S, acetyl choline will provide a fast and relatively physiological reconstruction of the pupil.

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<i>Design:</i>	<i>Schlosshardt</i>
<i>Overall length:</i>	<i>7.5 mm</i>
<i>Pupil size:</i>	<i>Ø 5.0 - 6.0 mm</i>
<i>Thickness:</i>	<i>0.15 - 0.60/0.90 mm PMMA CQ UV-Abs.CFM</i>
<i>First Production:</i>	<i>1994</i>

Morcher GmbH
 Kapuzinerweg 12
 D-70374 Stuttgart
 Tel. 07 11/9 53 20-0
 Fax 07 11/9 53 20-80
 info@morcher.com
 www.morcher.com