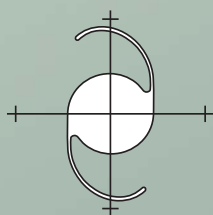
CAPSULAR TENSION RINGS

MORCHER[®] IOL

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| | |
|--------|--------|
| Typ 2 | 9,2mm |
| Typ 2A | 10,2mm |

Type 2A

The Capsular Ring is currently available in two sizes:

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Foldable Capsular Ring

The foldable ring according Prof. Dick, MD, made of the combimaterial PMMA/BioComFold, for even and circumferential stabilization of the capsular bag and reduction of secondary cataract.

Data:
One piece foldable Capsular Ring consisting of two polymerized materials in the proven „One Piece Two Material“ Technology (PMMA/BioComFold).

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Implantation

- Easy implantation with all available injector systems or forceps

- Folding of the rings:

- Single folding (the leading part opens first)
- Double folding (the ring opens and spreads evenly)

Advantage double folding:

The implantation of the ring puts no further pressure on the bag or the zonules, due to the ring's even opening and spreading

- Sizes:

Typ 2: The ring should only be used by small bulbus under 21mm of length

Typ 2A: The ring must only be used in bulbus' with length over 21 mm

Advantages Foldable Capsular Ring:

- Reduction of secondary cataract by "sharp edge", and closed ring design (360°)
 - Inhibition of capsular bag shrinkage
 - Minimizes or prevents capsular folds by expanding the bag circumferentially
- Due to the design and the implantation technique the ring does not stress the bag or the zonules
 - Reduces or prevents cloudiness of the anterior capsular wall, which facilitates fundus visualisation (e.g. on patients with retina problems)
- Potentially complicated surgical conditions
 - capsular bag distortion
- To simplify the implantation of foldable IOLs

Indications:

- Defective or partly missing zonules
 - Luxation of IOLs
 - Zonulolysis
 - Pseudoexfoliation
 - Marchesani Syndrome
- To stabilize the capsule in case of severe myopia

Contraindications:

- Type 2A: Bulbus under the length of 21 mm
- Zonula damage larger than 4 clock hours
 - Patients under the age of 1 year
 - Chronic uveitis
 - Progressive eye diseases (diabetic retinopathy, uncontrolled glaucoma)
- Preoperative complications prior to cataract surgery (vitreous body prolapse, hemorrhage)
- Patients with perforated or damaged capsules

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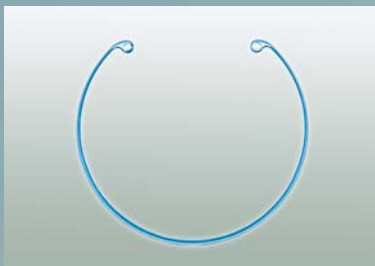
Capsular Tension Ring

The necessity in modern cataract surgery
Morcher Capsular Tension Rings are
manufactured from one-piece PMMA
and stabilize the capsule in cases of
damaged or missing zonules.

Series 14

The Capsular Tension Ring is
available in three different sizes:

| | |
|----------|---------|
| Type 14 | 10 mm |
| Type 14C | 11 mm |
| Type 14A | 12 mm * |

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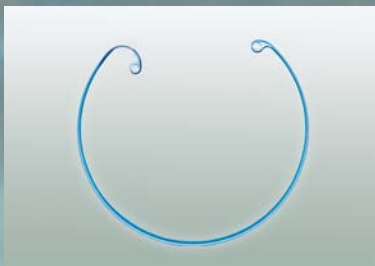
Series 13

The Capsular Tension Ring is
available in three different sizes:

| | |
|----------|---------|
| Type 13 | 10 mm |
| Type 13A | 11 mm |
| Type 13B | 12 mm * |

*size suitable for myopic eyes

**All Capsular Tension Rings can be implanted with forceps (MacPherson) or via injector.
We recommend the injector made by Geuder in Heidelberg, Germany: Geuder, G-32960**

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The bent eyelet of the ring significantly reduces the
danger of „impallment“, i.e. bulging the capsular
bag and, consequently, extending any zonula damage

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Advantages of the Capsular Tension Rings:

- circular expansion of the capsular bag
- stabilization of the capsular bag
- on small zonula damages, no sclera fixation is
necessary, i.e. no special IOL is required
- reduces the risk of damaging the capsule or
zonules during surgery
- better centration of the IOL, also in cases of
future capsular bag shrinkage
- potentially complicated surgical conditions
- to simplify the implantation of foldable IOLs

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Indications:

- defective or partly missing zonules
- luxation of IOLs
- zonulolysis
- pseudoexfoliation
- Marchesani Syndrome
- to stabilize the capsule in case of severe myopia

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Contraindications:

- zonula damage larger than 4 clock hours
- patients under the age of 1 year
- children 12 years of age or younger (only valid for U.S.A.)
- chronic uveitis
- progressive eye diseases
(diabetic retinopathy, uncontrolled glaucoma)
- preoperative complications prior to cataract surgery
(vitreous body prolapse, hemorrhage)
- patients with perforated or damaged capsules

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| | |
|--------|-------|
| Typ 1G | 10 mm |
| Typ 1L | 11 mm |

Cionni Capsular Tension Rings

Capsular Tension Rings, manufactured from one piece PMMA, to stabilize the capsular bag in cases of damaged or missing zonules. This design can also be sclera fixated and remains in the eye.

Types 1G and 1L

The Cionni Capsular Tension Ring is available in two different sizes:

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Type 2C

The Capsular Tension Ring Type 2C with the potential for scleral fixation is similar to Type 1L. This type is specially designed with the fixation hook on the other side of the ring which permits implantation with a special injector.

Note: For Type 2C we recommend the G-32960 Injector from Geuder/Heidelberg. Types 1G, 1L, 2L and Type 2S cannot be implanted with an injector! For fixation, we recommend 9.0 double armed prolene sutures with spatula needles.

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Advantages of the Cionni Rings:

- circular expansion of the capsular bag
- stabilization of the capsular bag
- reduces the risk of damaging the capsule or zonules during surgery
- better centration of the IOL, also in cases of future capsular bag shrinkage
- potentially complicated surgical conditions
- to simplify the implantation of foldable IOLs

Indications:

- zonula damage larger than 4 clock hours
- defective or partly missing zonules
- luxation of IOLs
- zonulolysis
- pseudoexfoliation
- Marfan's-Syndrome
- to stabilize the capsule in case of severe myopia

Contraindications:

- patients under the age of 1 year
- children 12 years of age or younger (only valid for U.S.A.)
- chronic uveitis
- progressive eye diseases (diabetic retinopathy, uncontrolled glaucoma)
- preoperative complications prior to cataract surgery (vitreous body prolapse, hemorrhage)
- patients with perforated or damaged capsules

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Ahmed Capsular Tension Segment

The Ahmed Capsular Tension Segment (CTS) is an alternative to capsule supporting hooks. It is designed for safe placement prior to phacoemulsification, with a fixation eyelet that can be secured with a modified iris hook and/or sutured to retract the lens over the area of zonular dialysis. The CTS can be left in-the-bag after lens removal for suture fixation to the sclera, providing long-term support over the area of zonulolysis. In cases of profound circumferential zonular weakness, 2 CTS's may be placed 180° apart.

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Advantages Ahmed Capsular Tension Segment :

- reduces or avoids IOL subluxation in cases of profound zonular dialysis
 - stabilizes the capsular bag
- reduces the risk of damaging the capsule or zonules during surgery
- better centration of the IOL, also in cases of future capsular bag shrinkage
 - potentially complicated surgical conditions
- to simplify the implantation of foldable IOLs

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Indications:

- zonula damage larger than 4 clock hours
 - defective or partly missing zonules
 - luxation of IOLs
 - zonulolysis
 - pseudoexfoliation
 - Marfan's-Syndrome
- to stabilize the capsule in case of severe myopia

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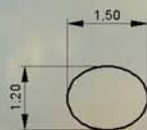
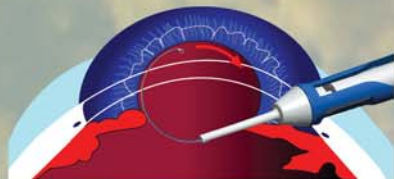
Contraindications:


- patients under the age of 1 year
- children 12 years of age or younger (only valid for U.S.A.)
 - chronic uveitis
 - progressive eye diseases (diabetic retinopathy, uncontrolled glaucoma)
- preoperative complications prior to cataract surgery (vitreous body prolapse, hemorrhage)
- patients with perforated or damaged capsules

EYEJET CTR

SINGLE USE PRE-LOADED CAPSULAR TENSION RING

Dimensions of the EYEJET tip

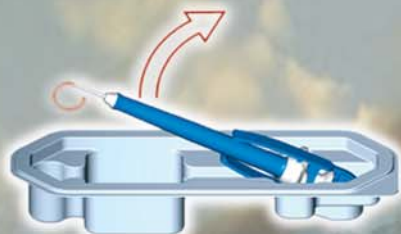1. Remove from package

2. Push the clamp backwards

3. Remove the clamp

4. Implantation of the Capsular Ring

EyeJet Capsular Tension Rings

The easy way to implant

This preloaded CTR injector is available in three different sizes:

Type 14 10 mm
Type 14C 11 mm
Type 14A 12 mm *

*size suitable for myopic eyes

This preloaded CTR injector is available in three different sizes:

Type 13 10 mm
Type 13A 11 mm
Type 13B 12 mm *

*size suitable for myopic eyes

Type 19D

9 mm to 10,5

Type 10C

11 mm

MORCHER[®] IOL.

MORCHER[®]IOL

Version:2009-05-01 Tsa

Morcher GmbH Kapuzinerweg 12 D-70374 Stuttgart Tel.: +49 (0)7 11/9 53 20-0 Fax: +49 (0)7 11/9 53 20-80 e-mail: info@morcher.com www.morcher.com